

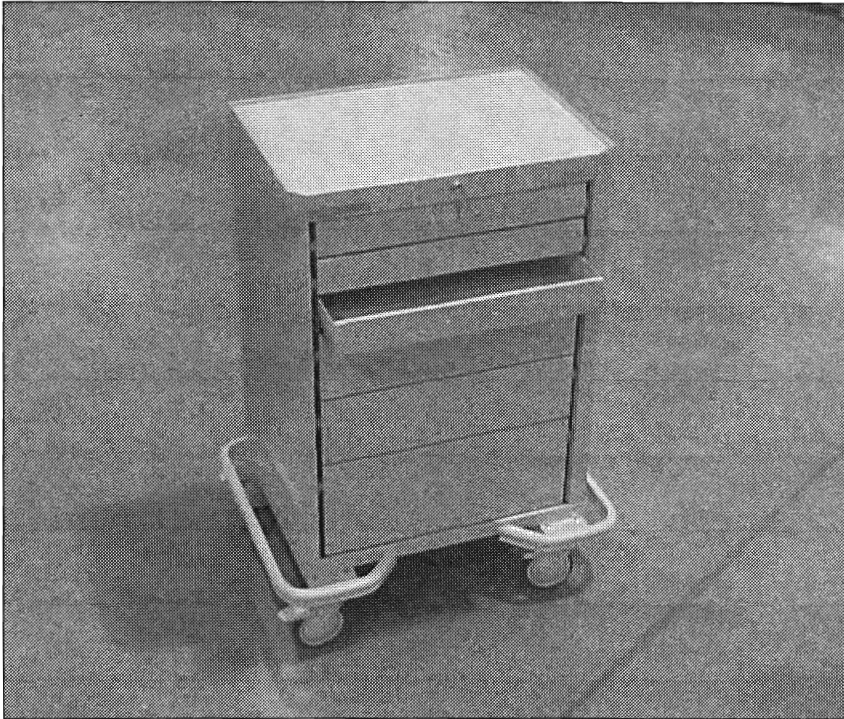


**ARMSTRONG MEDICAL INDUSTRIES INC.
A-SMART STEEL CARTS**

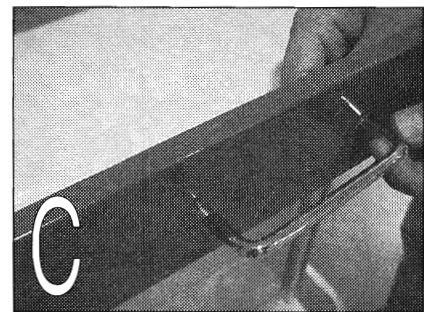

Steel Cart shown as removed out of box.

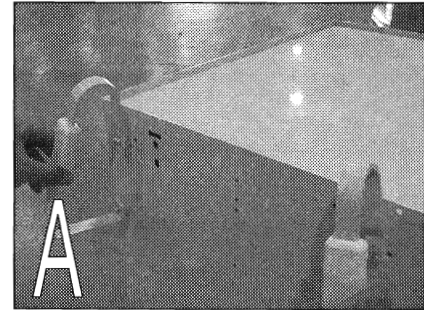

A: Handle Mounting

Mount (2) handles using (4) 1/4-20 x 1/2 machine screws on same side as locking casters.

B: Drawer Front Assembly


Match drawer front height to drawer body. Slip top of drawer front over top of body, then snap at bottom.

C: Drawer Pulls

Use (2) machine screws supplied with drawer pulls and assemble to drawers.


CARTS VARY IN HEIGHT, DRAWER
ARRANGEMENT AND DRAWER LOCKING STYLE

DRAWER REMOVAL

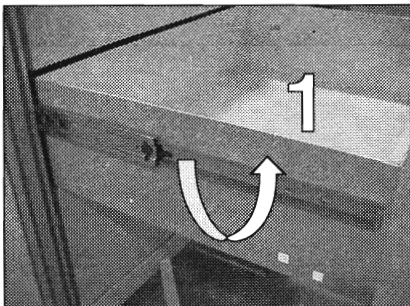


1. Pull drawer forward lift plastic lever up on left & down on right & pull drawer out.

SLIDE REMOVAL

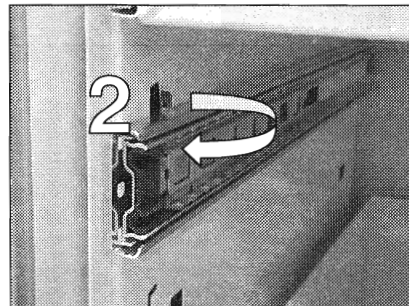

2. Bend slide retainment tab in. Lift slide then slide forward & out.

RUBBER STRIP LOCATIONS

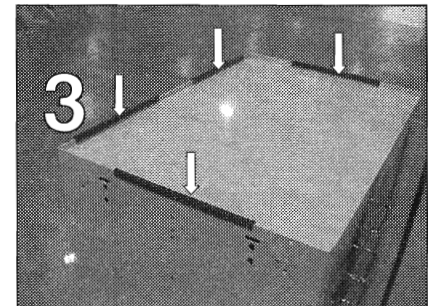

3. Use only with plastic top.

Armstrong Medical Industries Inc.

575 Knightsbridge Parkway • Post Office Box 700 • Lincolnshire, IL 60069-0700
800/323-4220 • Local: 847/913-0101 • Fax: 847/913-0138
www.armstrongmedical.com

REV. A
BYAM 307

1024