

Equipment Maintenance Checklist

✓ After Each Patient

Clean (w/ultrasonic) & Sterilize Syringe Tips

Clean, Lubricate, & Sterilize Handpieces

Flush Waterlines (handpiece, air/water syringe, scaler, etc)

Replace All Barriers (clean / disinfect touch and transfer surfaces not covered by barriers)

✓ Daily or As Needed

ICX Waterline Treatment

(add one tablet to an empty water bottle before refilling)

Clean Vacuum Lines

(recommended daily for general dentistry, after each patient for oral surgery)

✓ Weekly

Clean HVE & Saliva Ejector Valves

Inspect & Maintain Solids Collector (twice weekly)

Remove & Clean Dental Light Shield with Mild Soap & Water

✓ Monthly

Monitor Water Quality & Maintain As Needed

Maintain Oil Collector

Lubricate HVE & Saliva Ejector O-rings

✓ Yearly

Replace & Lubricate Syringe O-rings

Lubricate Syringe Button Assemblies

Replace & Lubricate HVE & Saliva Ejector O-rings

