

Lisa MB17 Sterilizer

WEEKLY STERILIZER MAINTENANCE

1. Door Seal – Clean door seal and port hole with a lint free cloth and alcohol or non-abrasive detergent.
2. Chamber – Clean the inside of chamber with a damp sponge moistened with a detergent or scouring agent if necessary. Always rinse the chamber with a damp sponge to remove traces of detergent.
Note: Do not use disinfectants to clean the chamber.
Note: Do not bend the temperature sensor at chamber bottom.
3. External Surfaces – Clean with a damp sponge and mild detergent. Do not use scouring agents or abrasive products.
4. Main water tank disinfection –
 - a. Open service door at lower right side of LISA, insert drain tube into the ports for clean and dirty water tanks and discard all water from both tanks.
 - b. Add 7 ounces of 90% isopropyl alcohol to main water reservoir, then add distilled water until full. **Do not run a cycle at this time.**
 - c. Allow the solution to sit for 30 minutes, drain water/alcohol solution and refill with distilled water, approximately 3 quarts.
 - d. Run an empty cycle.

AS REQUIRED STERILIZER MAINTENANCE

1. The selection screen will tell you when it is time for performing other maintenance tasks.
 - a. Every 400 Cycles – Replacing the filter (Page 34 Owner's Guide)
 - b. Every 1,000 Cycles – Replacing the door gasket (Page 34 Owner's Guide)
 - c. Every 4,000 Cycles – Service by Authorized Technician (See Page 35 Owner's Guide)
2. After replacement of the filter and door gasket, make sure to reset LISA's cycle counter. (See Page 20 Owner's Guide)

MESSAGES AND ALARMS

1. A microprocessor continuously analyzes the parameters of each cycle. A message or alarm may be displayed to inform you of any requirements or potential problems. Please refer to page 27 and 28 of the Owner's Guide. A description and instructions are given for messages and alarms are given on these two pages. Follow the instructions that are given.