



CODE	DESCRIPTION	QTY
A	Caster	3
B	Base	1
C	Tube # 1	1
D	Tube # 2	2
E	Top Ring	1
F	Connecting Rod	1
G	Lid	1
H	Foot Pedal	1
I	Screw & Nut Set	1
	Spanner Wrench	2

### *Assembly Instructions*

1. *Insert casters (A) into base*
2. *Insert tube #1 (C) with counter-weight into the front base end projection & tubes (D) into the other two base projections.*
3. *Attach top ring with attached lid to top ends of tubes (C) and (D).*
4. *Screw connecting rod (F) to the lid (G) and the rear of the foot pedal (H) with the screw & nut set (I) with the provided spanner wrenches.*