

# INSTRUCTION SHEET


# BEDSIDE CARTS



1. Use key in bag to unlock.
2. Open second drawer.
3. Attach rubber bumpers to bottom corners.
4. Use rubber strips when using plastic top. (2) at rear edge and (1) on left and right edge.




**Drawer Front Assembly:** Match drawer front height to drawer body. Slip top of drawer front over top of body, then snap at bottom.

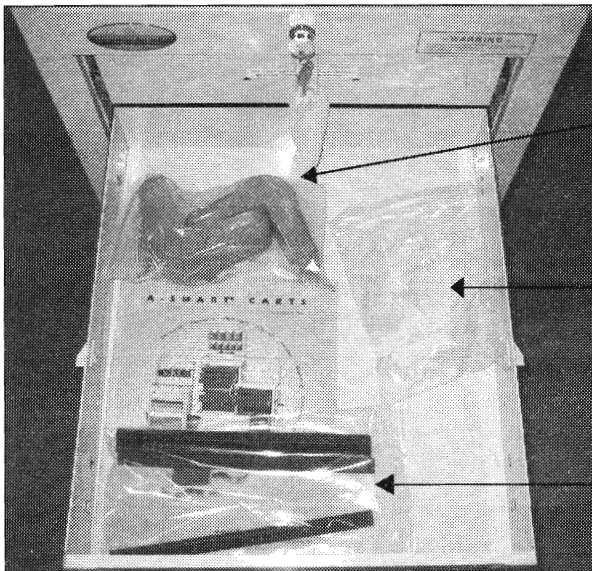

Rubber Bumpers


Handles

Rubber Strips

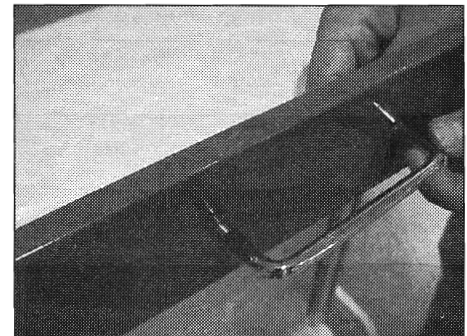

**Drawer Pulls:** Use (2) machine screws supplied with drawer pulls and assemble to drawers.

## RUBBER STRIP LOCATIONS

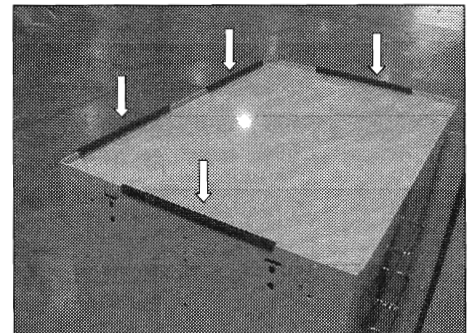

Use only with plastic top.

CARTS VARY IN HEIGHT, DRAWER  
ARRANGEMENT AND DRAWER LOCKING STYLE

### **Armstrong Medical Industries Inc.**

575 Knightsbridge Parkway • Post Office Box  
700 • Lincolnshire, IL 60069-0700

800/323-4220 • Local: 847/913-0101 • Fax: 847/913-0138  
www.armstrongmedical.com