

CUSTOM COMFORT MEDTEK

Draw-Med Chairs

Models:

1201-L, 1201-S, 1201N-L, 1201N-S, 1201-LXT, 1201-SXT, 1201N-LXT, 1201N-SXT, 1201-LU,
1201-SU, 1201-LU/XT, 1201-SU/XT, 1201-LXL, 1201-SXL, 1201-LXL/XT, 1201-SXL/XT,
1201-SXXL, 1201-SXXL/XT

Assembly/Usage Instructions

Insert Leg Levelers-

- Insert levelers into holes located on bottom of chair and adjust accordingly.

Attach Seat-

- When chair is equipped with a molded plastic seat- place the seat onto chair frame. Tighten seat to frame with the supplied nuts using a 7/16" socket or wrench.
- When chair is equipped with upholstered seat and back- seat and back are pre-attached for you.

Attach Arm Adjusters-

- Attach one arm adjuster on each side of chair with adjuster handles facing up and towards front of chair using a 7/16" socket or wrench.

Install Arms-

- Insert arm posts into arm adjuster. If chair has the optional "flip-up" arm, place flip-up arm on right side of chair. Place straight arm on left side of chair.

Usage Instructions

Arm Adjustment-

- To adjust armrests loosen the black adjuster handle and move the arm up/down or left/right. Re-tighten handle to secure arm into position. The black adjuster handle position can be adjusted by pressing in gray handle button and rotating handle.
- Adjust angle of the swing arm by rotating adjustment knob underneath armrest.