



Assembly

1. Using (2) 10-32 x 1/2" screws and (2) nuts mount glove support member (*Item #1*) onto body (*Item #2*).
2. Slide apron support rods (*Item #3*) through holes in column of body. Lock into position by placing a retaining ring, on each side of the column, into the groove of the support rod.
3. Secure unit to wall with bottom of body (*Item #2*) on the floor.
4. Attach glove holders (*Item #4*) to glove support member (*Item #1*) using the remaining (4) 10-32 x 1/2" screws and (4) nuts.

Specifications

- Dimensions: 30" wide x 69" height
- Shipping Weight: 25 lbs.
- Body made of heavy gauge steel finished in grey enamel. Glove brackets and apron support rods are made from anodized aluminum.
- Accommodates 5 Aprons and 4 Gloves.
- Unit is designed so weight is supported by floor.