KENDALL**IN-ROOM*****Sharps Disposal System****MOUNTING / INSTALLATION INSTRUCTIONS****INSTRUCTIONS DE MONTAGE • INSTRUCCIONES DE MONTAJE • INSTRUÇÕES DE MONTAGEM**

- For IN-ROOM* SHARPS Disposal System #85301H, 851608, 851609, 8556H, 85521H, 8516U, 85161H, 85165H, 85301HN, 8559, 85401H, 88361H
- Pour le Système d'élimination d'objets tranchants IN-ROOM* n° 85301H, 851608, 851609, 8556H, 85521H, 8516U, 85161H, 85165H, 85301HN, 8559, 85401H, 88361H
- Para el sistema de desecho de objetos punzantes IN-ROOM* n°85301H, 851608, 851609, 8556H, 85521H, 8516U, 85161H, 85165H, 85301HN, 8559, 85401H, 88361H
- Para Sistemas de Eliminação de Dispositivos Cortantes IN-ROOM* #85301H, 851608, 851609, 8556H, 85521H, 8516U, 85161H, 85165H, 85301HN, 8559, 85401H, 88361H

See reverse side for additional installation instructions.

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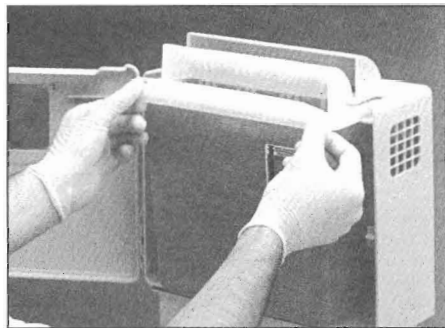
Wear the necessary safety equipment when installing sharps disposal systems.

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- **Standardize the location of sharps containers or pharmaceutical waste containers as much as possible.** Locate containers as close as possible to the area of use, keeping them out of the way of carts, beds and patient care objects. Special use areas such as Pediatrics, elderly/confused patient areas or high general public traffic areas raise additional safety issues. Individual hospital policies should be reviewed when choosing locations for containers.

Containers must be placed at a height that permits the user to clearly see the top of the container so that inadvertent contact with the contents of the container can be avoided and the level of the container contents can be safely determined.

In all events, the user must be able to visualize the line of disposal.

Please be aware that the results of unintended contact with infectious contents of sharps containers include injury or transmission of serious and potentially life threatening disease.

Don't place anything on top of the wall enclosure or container.

Example: Glove boxes, tissue boxes or used syringes.

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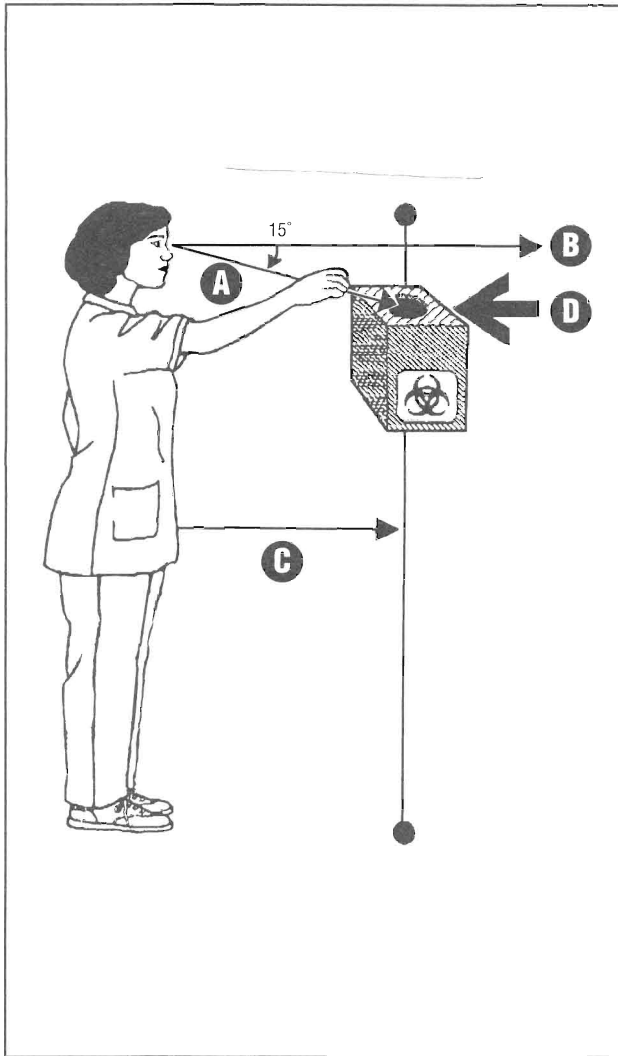
* Trademark of Sherwood Services AG

Covered by one or more of the following patent numbers: Patent No. 4,375,849; 1,288,402; des. 307,741; 5,211,303; des. 335,372; re. 33,373; des. 335,373

• **ERGONOMIC INSTALLATION HEIGHT FOR A WALL-MOUNTED WORK STATION.**

- A. Line of Sight
- B. Eye Level = 57 inches (145 cm)
- C. Thumb tip reach - 11 - 19 inches (28 - 48 cm)
- D. **An ideal standing installation height for a fixed sharps disposal container is 52 - 56 inches (132 - 142 cm). This height will comfortably accommodate 95% of all adult female workers.**

Illustration from: DHHS (NIOSH) Publication No. 97-111, "Selecting, Evaluating, and Using Sharps Disposal Containers."

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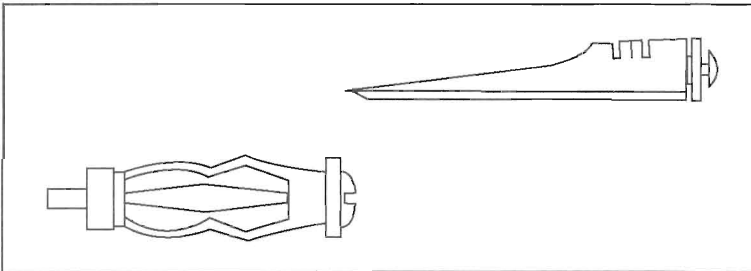
Ilustração de: DHHS (NIOSH) Publication No. 97-111 "Selecting, Evaluating and Using Sharps Disposal Containers."

■ **NEW INSTALLATION:**

- Select the appropriate fasteners for the type of mounting surface chosen (plaster, wall board, etc.). A minimum of four fasteners is recommended.
- Place the cabinet at a height that will allow the user to visualize the top of the container. Please refer to previous page for guidelines.
- Use mounting slots to secure cabinet to a stable vertical surface.
- Make sure cabinet is level, secure and flush with the surface on which it is mounted.

■ **EXISTING INSTALLATION:**

- This cabinet can also be mounted using existing holes by removing existing bracket or cabinet and aligning the new cabinet with the holes and refastening to mounting surface.
- Additional slots on wall enclosure may be used to adjust height without drilling new holes.
- Follow guidelines above ("New Installation").

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